


10THERA

10THERA is '2 Line Irradiating' HIFU.



'HIFU'(High Intensity Focused Ultrasound) is a non-invasive lifting method, which delivers ultrasound energy to target layer without damaging the skin surface. HIFU generates 53~80 degree heat on the SMAS and dermal layer to form thermal coagulation point and regenerates collagen fiber and elastic fiber for lifting effect.

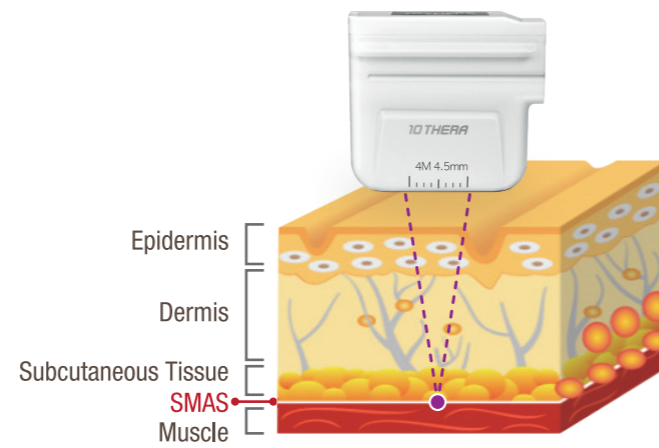

10THERA is recommended for people who:

- ▶ are looking for natural-looking lifting
- ▶ are worried about skin laxity and looseness
- ▶ are concerned about sagging cheeks and double chin
- ▶ are interested in a treatment without damage on the skin surface
- ▶ want a treatment without visible marks
- ▶ want HIFU treatment with shorter treatment time

2 Line Irradiating HIFU

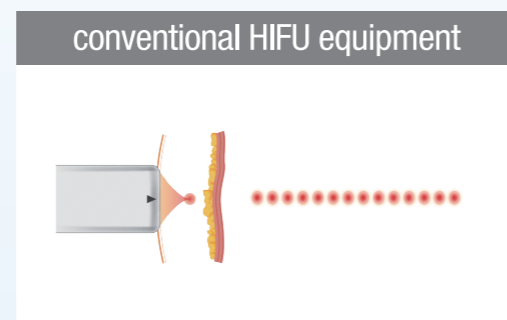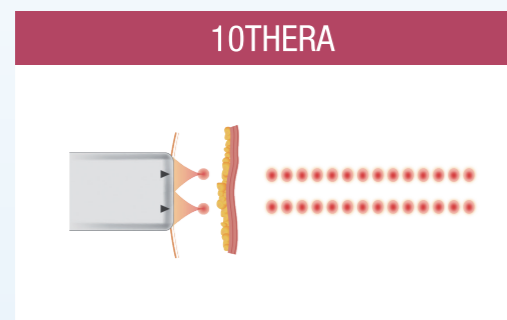

“ 200 shots of 10THERA similar to 400 shots of other 1 line HIFU equipment ”

Irradiates 2 lines simultaneously, making the thermal coagulation zone twice as big. Based on the same number of shots, the treatment time is reduced to 1/2.

Cartridges

	7Mhz 3.0mm	4Mhz 4.5mm
Type		
Treatment area	chin, cheek, neck	cheek

Specification

Model	10THERA
Cartridge	4Mhz, 4.5mm 7Mhz, 3.0mm
Spacing	1.5 ~ 2.0mm
Length	5.0 ~ 25.0mm
User Interface	15" Touch LCD
Dimension	W 550 x D 520 x H 1200mm
Weight	30 Kg
Electrical Requirement	100-240V~, 50/60Hz, 300VA



This product is of aesthetic device of medical grade, please refer to the precautions and treatment instructions before using it.

Q&A

1. What can I expect from 10THERA treatment?

HIFU treatment is used for skin tightening, face lifting and reducing fine lines and wrinkles. The treated area will show improvement of elasticity and lifting, and collagen regeneration.

2. When will you see the results?

You can see the lifting effect right after the treatment. However, this is a temporary phenomenon in which the skin and muscles are surprised and tightened by strong ultrasound. The actual and desired effect will take place when the damaged tissue is restored, tightening up your skin.

3. How long 10THERA last and how often should I take it?

Usually, you can see the most vivid results 1~2 months later and it will last up to 6 months. Recommended intervals between treatments are every 2~3 months but it can vary from person to person.

4. Is there any discomfort during the treatment?


The pain may feel different depending on the treated area, in general, patients report experiencing mild pricking and tingling during the treatment. If you have sensitive skin, the procedure can be performed with a lower energy level.

5. What are the precautions after the HIFU treatment?

There are no special precautions, you will be able to go back to your normal activities after the treatment. Because it could reduce the effect, we recommend you to refrain from getting acupuncture massage, drinking alcohol, and smoking.

6. Are there any side effects?

Since HIFU transmits ultrasonic heat energy deep into the skin, not to the surface, patients can return to daily life right after the procedure. If you have sensitive skin, mild skin redness can occur. If the cartridge is not tightly placed on the skin because of the movement of the patient, HIFU heat can be transferred to the upper dermis, which is not the correct target thus leading to blisters in linear form. This will naturally disappear in a week or so.

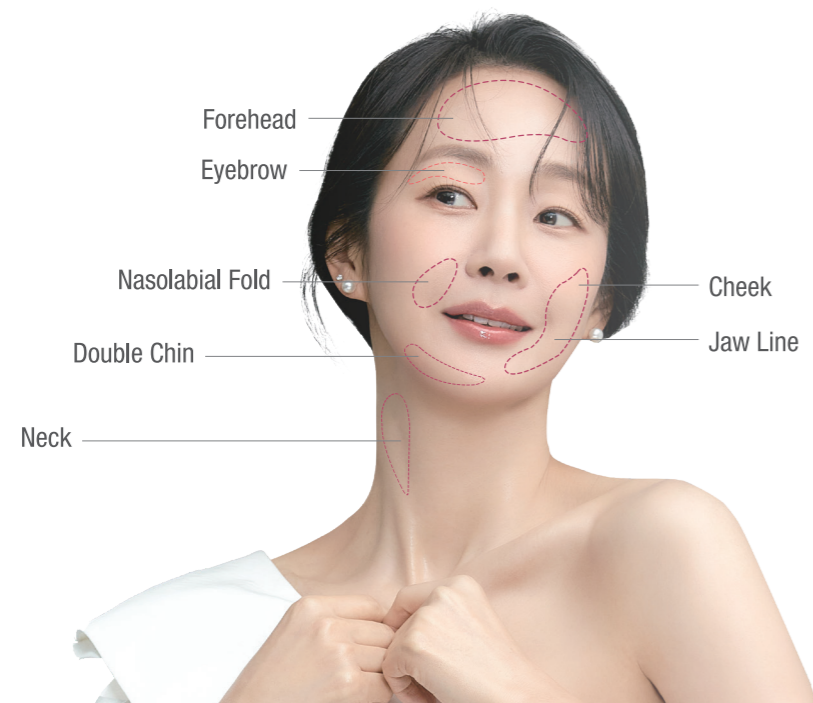

10THERA

2 Line HIFU

Feel the elasticity filling up from the deep inside of the skin
Improvement of skin elasticity / reducing fine lines / face shaping



Tentech Inc. 4F, 5F Namjeon Building, 326, Bongeunsa-ro, Gangnam-gu, Seoul, Korea
TEL. 02-6954-2203 | FAX. 02-3444-4999 | www.tenlaser.com

